



My FILLER
Revitalize

○ ○ ○ ○



Vendel utca, 11
1096 Budapest - Hungary
www.my-med.eu
info@my-med.eu



MESOTHERAPY
COMPLETE PROTOCOL OF USE



GENERAL DESCRIPTION OF LINE



The optimal post treatment results of **My Filler** generated great satisfaction among patients, which made **My Filler** the “first choice”.

The aesthetic medical professionals are confident with **My Filler** and it started to be used in a daily basis in their practice.

Such an achievement in the very competitive aesthetic field is a fact and thanks to **My Filler** superior quality and reliable distribution network, we win the trust the aesthetic professionals and patients day after day.

My Filler has several years of experience in injectable dermal filler technology (*Medical Device Class III*). With our scientific background and our commitment to offering top quality products, **My Filler** increased its anti-aging products range to better serve those patients and professionals who deposit their trust in our products.

My Filler's “New Life to the Skin” widens its application and creates **My Filler** “Revitalize”, the special line of Mesotherapy products.

My Filler Revitalize is a state-of-the-art range and complete portfolio of sterile topical cosmetics for professional use.



Revitalize

MyFILLER Revitalize Pure



Stretch marks, acne marks, scar marks and other skin imperfections are undesirable for anyone from an aesthetic point of view. The **Revitalize Pure** unique formulation has the perfect mixture and balance of collagen with hyaluronic acid. It increases the generation of new collagen fibres, repairs and reinforces the existing ones.

A repairing and restructuring treatment which was dreamed about, is now a reality...

REVITALZE PURE

Indication: For all skins that need to regenerate the texture and tonality.

Application:

Apply **Revitalize Pure** to the hands, face, neck and neckline, with electroporation techniques or radiofrequency. The microneedling technique can considerably speed up the results.

Month 1: 1 session every 15/30 days.
Month 2-3: 1 session every 30/60 days according to evolution.
Maintenance: 1 session every 60 days.

Contents:

- Sodium Hyaluronate 12mg/ml
- Hydrolyzed Collagen 5%

Packaging: 5 vials, 5 ml each.

Expiration: 3 years.



MYF500



Repair + Restructure



Steril



Topical Use



No Corticoids



Hands



Face and Neck



MyFILLER Revitalize Pure

A patch test must to be carried out 24 hours before treatment (with a sterile needle pierce the skin superficially behind the ear or on the forearm apply a tiny droplet of the serum and leave in place for minimum 24 hours). **If the skin reacts do not proceed with the treatment.**

REVITALIZE PROTOCOL



0

Cleanse the skin thoroughly and apply anaesthetic topical cream to the treatment area.

After **15-20 minutes** check if the area is numb. Remove topical cream with gauze and warm water. Double cleanse using a suitable skin cleansing solution such as a micellar cleanser & blot dry. Mask the treatment area using clean gauze.

1

Draw up **1ml** of **Revitalize Pure** per **10x10 cm** area using a sterile syringe and needle and then remove the needle (two serums may be mixed together forming a cocktail).



Amount of solution:
1ml per 10 x 10 cm

2

Deposit droplets of the serum onto the treatment area & spread evenly using a cotton bud or a sterile gloved finger. **0.5ml** at the first application.

0.5ml



3

Choose depth on **Revitalize Pen** to maximum depth (**0.25-1.5mm on the face** or **2-2.5mm on the body**) using the maximum speed. Begin to stamp the serum into the skin covering all areas using vertical & horizontal lines.



Depth:
Face: **0.25-1.5mm**
Body: **2-2.5mm**
Speed: **fast**

4

Reduce the depth and the speed of the **Revitalize Pen** to **50%** previously chosen. Deposit the remaining **0.5ml** serum onto the skin. Use the **Revitalize Pen** in a circular motion across the treatment area.




Depth: **reduce by 50%**
Speed: **medium**

5

Gently massage any remaining serum into the skin. Allow to dry naturally. Use a high SPF factor cream following treatment to protect from UV light.

Post treatment advice: Keep the treated area dry for 24 hours & always wear a high SPF cream.

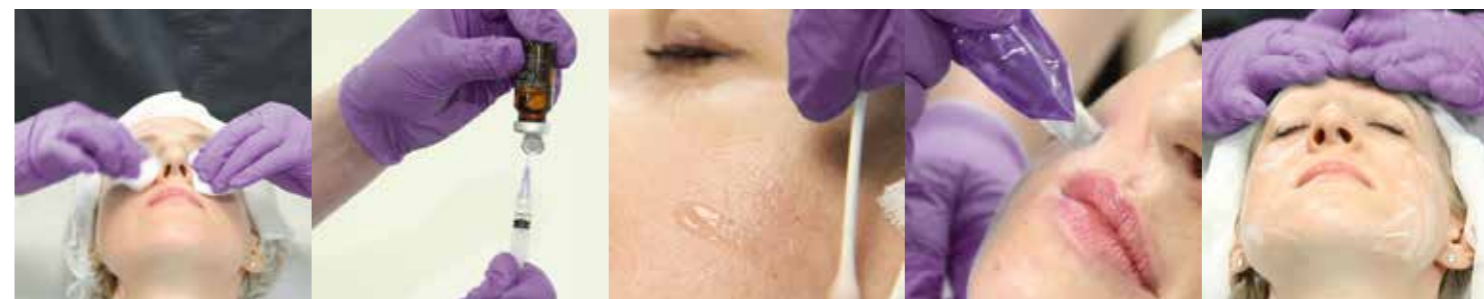

Revitalize



MyFILLER Revitalize Vitamins

Stress, sun over-exposure and wrong diet age the appearance. To keep it in good condition or recover it to a shiny and healthy skin, treat it correctly. **Revitalize Vitamins** special formula is composed of 70 essential ingredients for bringing back the skin's beauty lost somehow and to maintain it. Its composition contains: hyaluronic acid, vitamins, amino acids, minerals and coenzymes that improve and revitalize the state of the skin with an immediate result of healthier skin.

REVITALIZE VITAMINS

Indication: Revitalizing and illuminating.



MYF500

Application:

A weekly or a fortnightly **Revitalize Vitamins** application is recommended by therapy with microneedle and/or virtual mesotherapy by electroporation.

Month 1:

1 session every 7 days.

Month 2:

1 session every 15 days.

Maintenance:

1 session every 30 days.

Contents:

- Vitamin complex

Packaging:

5 vials, 5 ml each.

Expiration:

3 years.



Moisturizing + Nutrition



Steril



Topical Use



No Corticoids



Chest



Hands



Face and Neck



MyFILLER Revitalize Vitamins

A patch test must to be carried out 24 hours before treatment (with a sterile needle pierce the skin superficially behind the ear or on the forearm apply a tiny droplet of the serum and leave in place for minimum 24 hours). **If the skin reacts do not proceed with the treatment.**

REVITALIZE PROTOCOL



0

Cleanse the skin thoroughly and apply anaesthetic topical cream to the treatment area.

After **15-20 minutes** check if the area is numb. Remove topical cream with gauze and warm water. Double cleanse using a suitable skin cleansing solution such as a micellar cleanser & blot dry. Mask the treatment area using clean gauze.

1

Draw up **1ml** of **Revitalize Vitamins** per **10x10 cm** area using a sterile syringe and needle and then remove the needle (two serums may be mixed together forming a cocktail).



Amount of solution:
1ml per 10 x 10 cm

2

Deposit droplets of the serum onto the treatment area & spread evenly using a cotton bud or a sterile gloved finger. **0.5ml** at the first application.



0.5ml

3


Choose depth on **Revitalize Pen** to maximum depth (**0.25-1.5mm on the face** or **2-2.5mm on the body**) using the maximum speed. Begin to stamp the serum into the skin covering all areas using vertical & horizontal lines.



Depth:
Face: **0.25-1.5mm**
Body: **2-2.5mm**
Speed: **fast**

4

Reduce the depth and the speed of the **Revitalize Pen** to **50%** previously chosen. Deposit the **remaining 0.5ml** serum onto the skin. Use the **Revitalize Pen** in a circular motion across the treatment area.

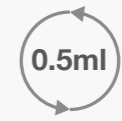

0.5ml




Depth: **reduce by 50%**
Speed: **medium**

5

Gently massage any remaining serum into the skin. Allow to dry naturally. Use a high SPF factor cream following treatment to protect from UV light.

Post treatment advice: Keep the treated area dry for 24 hours & always wear a high SPF cream.

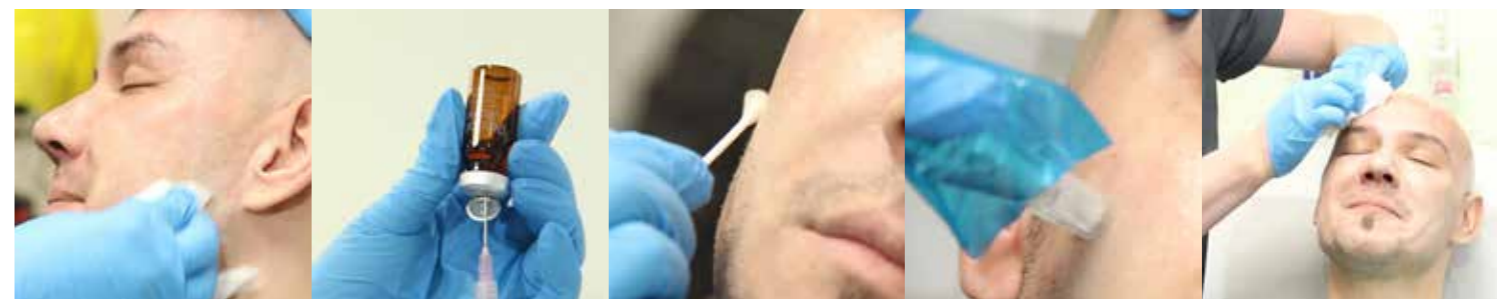

Revitalize

MyFILLER Revitalize Slim



Poor diet, lack of exercise and stress are the main reasons of excess fat accumulation. It has become one of the main requests for an aesthetic treatment. **Revitalize Slim** new lipolytic and anti-cellulite formula is composed of an insulin-like growth factor, phosphatidylcholine, sodium deoxycholate, carnitine and trace elements.

Effectiveness from the first session...

REVITALIZE SLIM

Indications: Localized fat accumulation spot as tummy, buttocks, back of the arm, thighs and double chin.

Application:

A weekly or a fortnightly application of **Revitalize Slim** is recommended through therapy with microneedle and/or virtual mesotherapy for electroporation.

Month 1: 1 session every 15/30 days.
Month 2: 1 session every 30/60 days according to evolution.

Contents:

- Phosphatidylcholine 2.5%
- Sodium Deoxycholate 1.5%,
- Carnitine 4%

Packaging: 5 vials, 10 ml each.

Expiration: 3 years.



MYF500



Repair + Restructure



Steril



Topical Use



No Corticoids



Hands



Face and Neck

MyFILLER Revitalize Slim

A patch test must to be carried out 24 hours before treatment (with a sterile needle pierce the skin superficially behind the ear or on the forearm apply a tiny droplet of the serum and leave in place for minimum 24 hours). **If the skin reacts do not proceed with the treatment.**

REVITALIZE PROTOCOL

0

Cleanse the skin thoroughly and apply anaesthetic topical cream to the treatment area.

After **15-20 minutes** check if the area is numb. Remove topical cream with gauze and warm water. Double cleanse using a suitable skin cleansing solution such as a micellar cleanser & blot dry. Mask the treatment area using clean gauze.

1

Draw up 1ml of **Revitalize Slim** per **10x10 cm** area using a sterile syringe and needle and then remove the needle (two serums may be mixed together forming a cocktail).



Amount of solution:
1ml per 10 x 10 cm

2

Deposit droplets of the serum onto the treatment area & spread evenly using a cotton bud or a sterile gloved finger. **0.5ml** at the first application.



0.5ml

3

Choose depth on **Revitalize Pen** to maximum depth (**0.25-1.5mm on the face** or **2-2.5mm on the body**) using the maximum speed. Begin to stamp the serum into the skin covering all areas using vertical & horizontal lines.



Depth:
Face: **0.25-1.5mm**
Body: **2-2.5mm**
Speed: **fast**

4

Deposit the remaining **0.5ml** serum onto the skin. Use the **Revitalize Pen** in a circular motion across the treatment area.



0.5ml



Depth:
Face: **0.25-1.5mm**
Body: **2-2.5mm**
Speed: **fast**

5

Gently massage any remaining serum into the skin. Allow to dry naturally. Use a high SPF factor cream following treatment to protect from UV light.

Post treatment advice: Keep the treated area dry for 24 hours & always wear a high SPF cream.



Revitalize

MyFILLER Revitalize Body



Body and facial flaccidity is a direct consequence of aging and/or weight variations. To maintain a firm skin, without wrinkles, we must be very consistent with a healthy diet and good hydration habits. **Revitalize Body** formula is composed of hyaluronic acid, organic compounds, DMAE and Dexpanpentol, which will reaffirm and restore the collagen fibres that support the skin, shaping them again, removing the skin sagging effect and flaccidity.

REVITALIZE BODY

Indications: Sagging or loose skin and dark under eye circles.

Application:

A weekly or a fortnightly application of **Revitalize Body** is recommended through therapy with microneedle and/or virtual mesotherapy for electroporation.

- Month 1: 1 session every 7 days.
- Month 2: 1 session every 15 days.
- Maintenance: 1 session every 30 days.

Contents:

- Methylsilanol Mannuronate 2.5%
- Dimethylaminoethanol Tartrate 1%

Packaging: 5 vials, 5 ml each.

Expiration: 3 years.



MYF500



Firming




Steril



Topical Use



No Corticoids

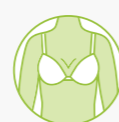

Chest



Neck



MyFILLER Revitalize Body

A patch test must to be carried out 24 hours before treatment (with a sterile needle pierce the skin superficially behind the ear or on the forearm apply a tiny droplet of the serum and leave in place for minimum 24 hours). **If the skin reacts do not proceed with the treatment.**

REVITALIZE PROTOCOL

0

Cleanse the skin thoroughly and apply anaesthetic topical cream to the treatment area.

After **15-20 minutes** check if the area is numb. Remove topical cream with gauze and warm water. Double cleanse using a suitable skin cleansing solution such as a micellar cleanser & blot dry. Mask the treatment area using clean gauze.

1

Draw up **1ml** of **Revitalize Body** per **10x10 cm** area using a sterile syringe and needle and then remove the needle (two serums may be mixed together forming a cocktail).



Amount of solution:
1ml per 10 x 10 cm

2

Deposit droplets of the serum onto the treatment area & spread evenly using a cotton bud or a sterile gloved finger. **0.5ml** at the first application.



0.5ml

3

Choose depth on **Revitalize Pen** to maximum depth (**0.25-1.5mm on the face** or **2-2.5mm on the body**) using the maximum speed. Begin to stamp the serum into the skin covering all areas using vertical & horizontal lines.



Depth:
Face: **0.25-1.5mm**
Body: **2-2.5mm**
Speed: **fast**

4

Reduce the depth and the speed of the **Revitalize Pen** to **50%** previously chosen. Deposit the **remaining 0.5ml** serum onto the skin. Use the **Revitalize Pen** in a circular motion across the treatment area.



Depth: **reduce by 50%**
Speed: **medium**

5

Gently massage any remaining serum into the skin. Allow to dry naturally. Use a high SPF factor cream following treatment to protect from UV light.

Post treatment advice: Keep the treated area dry for 24 hours & always wear a high SPF cream.





Revitalize

MyFILLER Revitalize Hair



Hair loss at an early age creates discomfort and embarrassment for anyone, man or woman.

Revitalize Hair acts in the prevention, re-densification, and stimulation of the hair folliculos matrix responsible for the hair, beard and eyebrow growth.

Moreover, **Revitalize Hair** acts at the dermal level on the fibroblasts situated next to the folliculos bulbs optimizing the development of the hair matrix.



MYF504

REVITALIZE HAIR

Indication: Nourishing the scalp and promoting hair growth.

Contents:

- DHQG } Redensyl
- EGCG }
- Glicina and Zinc

Packaging: 5 vials, 10 ml each

PROTOCOL OF TREATMENT:

PREPARATION

- A patch test must to be carried out 24 hours before treatment (with a sterile needle pierce the skin superficially behind the ear or on the forearm apply a tiny droplet of the serum and leave in place for minimum 24 hours). **If the skin reacts do not proceed with the treatment.**
 - Clean and sterilise the chosen area to be treated.
- Mix the content of 2 ampoules of **Revitalize Booster** into a vial of **Revitalize Hair**.
 - Shake until an homogeneous emulsion forms.








MyFILLER Revitalize Hair

HAIR REINFORCEMENT

One session per week for a maximum of 6 weeks.

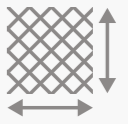


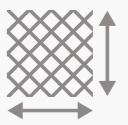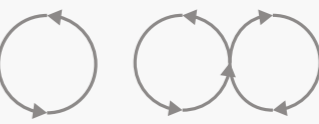

TREATMENT

- 1 Remove the needle and use the syringe as a pipette to deposit droplets of the solution onto the treatment area. Spread the solution using a cotton bud **0.5ml** of the solution per **10 by 10cm** area.  Amount of solution: **0.5ml** Treatment area: **10cm x 10cm**
- 2 Choose the depth on the revitalize pen up to **1mm** with the **highest speed** (in case of skin sensitivity use only **0.75mm**). Start working in straight lines horizontally and vertically.  Depth: **0.75-1mm** Speed: **fast** 
- 3 Apply another **0.5ml** of the solution at the same area.  Amount of solution: **0.5ml** Treatment area: **10cm x 10cm**
- 4 Work with the **Revitalize Pen** in a circular motion covering all the area with the **same depth and speed**.  Revitalize Pen depth: **0.75-1mm**

HAIR TRANSPLANT SUPPORT

One session per week for minimum 6 weeks.

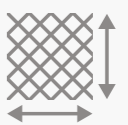TREATMENT

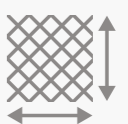- 1 Remove the needle and use the syringe as a pipette to deposit droplets of the solution onto the treatment area. Spread the solution using a cotton bud **0.5ml** of the solution per **10 by 10cm** area.  Amount of solution: **0.5ml** Treatment area: **10cm x 10cm**
- 2 Choose the depth on **Revitalize Pen** to **1mm** with **medium speed**. Start working in straight lines horizontally and vertically.  Depth: **1mm** Speed: **medium** 
- 3 Apply another **0.5ml** of the solution at the same area.  Amount of solution: **0.5ml** Treatment area: **10cm x 10cm**
- 4 Work with the **Revitalize Pen** in a circular motion covering all the area with the **same depth and speed**.  Revitalize Pen depth: **1mm**

BEARD

One session per week for a maximum of 6 weeks.

TREATMENT

1 Remove the needle and use the syringe as a pipette to deposit droplets of the solution onto the treatment area. Spread the solution using a cotton bud **0.5ml** of the solution per **10 by 10cm** area.




Amount of solution: **0.5ml**




Treatment area: **10cm x 10cm**


2 Choose the depth on **Revitalize Pen** to **0.75mm** with the **highest speed**. Start working in straight lines horizontally and vertically.



Depth: **0.75mm**



Speed: **fast**



3 Apply another **0.5ml** of the solution at the same area.





Amount of solution: **0.5ml**



Treatment area: **10cm x 10cm**

4 Work with the **Revitalize Pen** in a circular motion covering all the area with the **same depth** and **speed**.

Revitalize Pen depth: **0.75mm**

EYEBROW

• First 3 weeks: Revitalize Booster and Hair for one session per week.
• Second 3 weeks: Revitalize Hair for one session per week.

TREATMENT

1 Remove the needle and use the syringe as a pipette to deposit droplets of the solution onto the treatment area. Spread the solution using a cotton bud **0.5ml** of the solution per **10 by 10cm** area.




Amount of solution: **0.5ml**




Treatment area: **10cm x 10cm**


2 Choose the depth on **Revitalize Pen** to **0.5mm** with the **lowest speed**. Start working in straight lines horizontally.



Depth: **0.5mm**



Speed: **slow**



3 Apply another **0.5ml** of the solution at the same area.





Amount of solution: **0.5ml**



Treatment area: **10cm x 10cm**

4 Work with the **Revitalize Pen** in straight lines horizontally covering the same area with the **same depth** and **speed**.

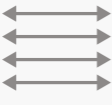Revitalize Pen depth: **0.5mm**

POST TREATMENT RECOMMENDATIONS

1. Do not touch the treated area.
2. Avoid skin/hair preparations for 24 hours.
3. Expect swelling redness and bruising (these are temporary).
4. If you experience undue discomfort contact your practitioner immediately.
5. Expect some mild irritation and bumps under the skin (this is normal and will disperse).
6. Keep the treated area dry for 24 hours.
7. Avoid alcohol, exercise and extreme temperature for 48 hours.
8. Stay out of the sun (cover the area if going outside).
9. Avoid sun-beds, saunas and steam rooms.
10. Gently wash with a mild shampoo after 24 hours.
11. Do not use abrasive combs or hairbrushes.
12. Contact your practitioner if you have any concerns following treatment.



Repair
+
Restructure



Sterile



External
Use



Hair

Following **Revitalize Hair** clinical trials microneedling techniques obtain the best aesthetic results with less trauma for the patient.

Revitalize Hair can be used with other mesotherapy techniques such as High Frequency (HiFu), Electroporation and Mesogun. Please note that other techniques may require a different setting. Refer to your training manual.

Precautions:

Topical use only, **DO NOT INJECT**. Avoid contact with eyes. In case of contact with eyes, rinse immediately with plenty of water. It is not recommended to use this product if you are allergic to any of its ingredients. Keep out of children's reach. Store under 25°C in a dry place

Do not expose to direct sunlight.

EXPLORE OUR COMPLETE MESOTHERAPY LINE:

A range of award winning serums:

Revitalize




www.myfiller.eu/revitalize

Revitalize



MyFILLER Revitalize White Skin

Mesotherapy allows to reach the deeper layers of skin. Depositing the serum, **Revitalize White Skin**, increasing freshness and glow to the skin, thanks to its special formulation.

The effectiveness of whitening Meso is much higher than long term topical whitening creams.

REVITALIZE WHITE SKIN



MYF505

Indications: Lighten skin tone, reduce skin spots, repair sun damage and protect against future damage from UV radiation.

Application: Revitalize White Skin protocol is recommended through laser, therapy with microneedle and/or virtual mesotherapy for electroporation.

Contra-indications: Do not use on Rosacea.

1st and 2nd Month: 8 applications (once a week).
If the final results achieved before the 8 applications, the protocol can be suspended. The process can speed up with special techniques (E.g.: Revitalize Booster).

Contents:

- Kojic Acid
- Retinol
- Ascorbic Acid (Vitamin C)

Packaging: 5 vials, 5 ml each.

Expiration: 3 years.



Non injectable



Whitening




Sterile



External Use



Body

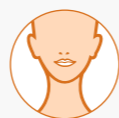

Face



MyFILLER Revitalize White Skin


A patch test must to be carried out 24 hours before treatment (with a sterile needle pierce the skin superficially behind the ear or on the forearm apply a tiny droplet of the serum and leave in place for minimum 24 hours). **If the skin reacts do not proceed with the treatment.**

REVITALIZE PROTOCOL



0 Cleanse the skin thoroughly and apply anaesthetic topical cream to the treatment area.
After **15-20** minutes check if the area is numb. Remove topical cream with gauze and warm water. Double cleanse using a suitable skin cleansing solution such as a micellar cleanser & blot dry. Mask the treatment area using clean gauze.

1 Draw up **1ml** of **Revitalize White Skin** per **10x10** cm area using a sterile syringe and needle and then remove the needle (two serums may be mixed together forming a cocktail).

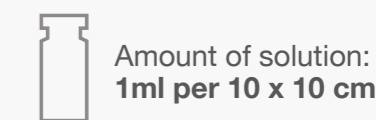


2 Deposit droplets of the serum onto the treatment area & spread evenly using a cotton bud or a sterile gloved finger. **0.5ml** at the first application.



3 Choose depth on **Revitalize Pen** to maximum depth (**0.25-1.5mm on the face** or **2-2.5mm on the body**) using the maximum speed. Begin to stamp the serum into the skin covering all areas using vertical & horizontal lines.



4 Reduce the depth and the speed of the **Revitalize Pen** to **50%** previously chosen. Deposit the **remaining 0.5ml** serum onto the skin. Use the **Revitalize Pen** in a circular motion across the treatment area.

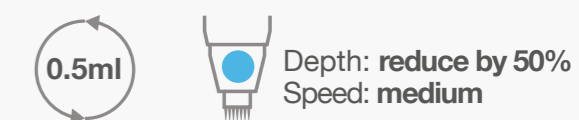

5 Gently massage any remaining serum into the skin. Allow to dry naturally. Use a high SPF factor cream following treatment to protect from UV light.

Post treatment advice: Keep the treated area dry for 24 hours & always wear a high SPF cream.



Revitalize

MyFILLER Revitalize Booster



Growth factors is naturally produced when a wound in our body requires a tissue reparation.

Revitalize Booster is a EGF (Epidermal Grow Factor) of last generation.

Booster is a protein composed by 53 amino-acids widely present in our skin. It increases keratinocyte proliferation of the corneal epithelium, endothelial cell and fibroblast (protein synthesis and glycosaminoglycans).



MYF506

REVITALIZE BOOSTER

Indication: It promotes dermal repair increasing its self-regeneration rate.

Booster is not injectable.

Contents:


4 ampoules with epidermal growth factor (EGF), an innovative protein that represents a revolution in beauty and regeneration treatments.

Packaging: 4 ampoules, 1 ml each.

PROTOCOL OF TREATMENT:

PREPARATION

- A patch test must to be carried out 24 hours before treatment (with a sterile needle pierce the skin superficially behind the ear or on the forearm apply a tiny droplet of the serum and leave in place for minimum 24 hours). **If the skin reacts do not proceed with the treatment.**
 - Clean and sterilise the chosen area to be treated.
- Mix the content of 2 ampoules of **Revitalize Booster** into a vial of **Revitalize Pure**.
 - Shake until an homogeneous emulsion forms.

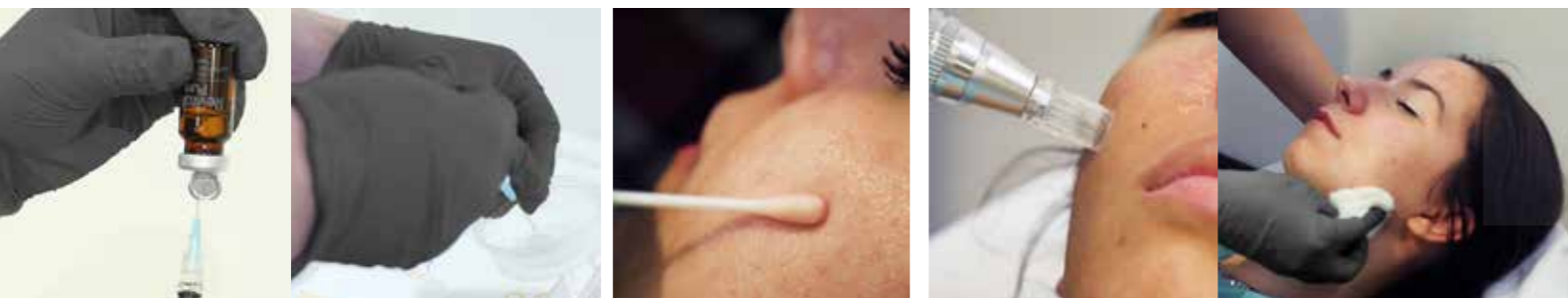

ACTIVE ACNE

One session per week for a maximum of 6 weeks.

TREATMENT

- 1 Apply with a cotton bud **0.5ml** of the solution at a **10cm x 10cm** area to be treated. Amount of solution: **0.50ml** Treatment area: **10cm x 10cm**
- 2 Choose the depth on **Revitalize Pen** to **0.50mm** (in case of skin sensivity use only **0.25mm**) and start work with it in straight lines horizontally and vertically at the area to be treated. Depth: **0.25-0.50mm** Speed: **slow**
- 3 Apply another **0.5ml** of the solution at the same area. Amount of solution: **0.50ml** Treatment area: **10cm x 10cm**
- 4 Work with the **Revitalize Pen** in **circle** or **"infinity" shapes** covering all the area with the **same depth**. Revitalize Pen depth: **0.25-0.50mm**

Complete the treatment with an acne cleanser, a serum regulating vial and a home treatment cream.

ACNE SCARS

One session per week for minimum 6 weeks.


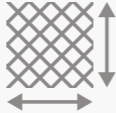







TREATMENT

- 1 Apply with a cotton bud **0.5ml** of the solution at a **10cm x 10cm** area to be treated. Amount of solution: **0.50ml** Treatment area: **10cm x 10cm**
- 2 Choose the depth on **Revitalize Pen** to **1-1.5mm** and start work with it in straight lines horizontally and vertically at the area to be treated. Depth: **1-1.5mm** Speed: **fast**
- 3 Apply another **0.5ml** of the solution at the same area. Amount of solution: **0.50ml** Treatment area: **10cm x 10cm**
- 4 Work with the **Revitalize Pen** in **circle** or **"infinity" shapes** covering all the area with the **same depth**. Revitalize Pen depth: **1-1.5mm**

ANTI AGE

One session per week for a maximum of 6 weeks.

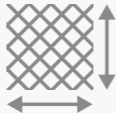
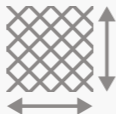







TREATMENT

- 1 Apply with a cotton bud **0.5ml** of the solution at a **10cm x 10cm** area to be treated.  Amount of solution: **0.50ml**  Treatment area: **10cm x 10cm**
- 2 Choose the depth on **Revitalize Pen** to **0.75mm** and start work with it in straight lines horizontally and vertically at the area to be treated.  Depth: **0.75mm**  Speed: **fast** 
- 3 Apply another **0.5ml** of the solution at the same area previously chosen.  Amount of solution: **0.50ml**  Treatment area: **10cm x 10cm**
- 4 Work with the **Revitalize Pen** in **circle** or **"infinity"** shapes covering all the area with the **same depth**.   Revitalize Pen depth: **0.75mm**

REGENERATOR


• First 3 weeks: Revitalize Booster for one session per week.
• Second 3 weeks: Revitalize Vitamins for one session per week.

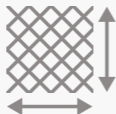TREATMENT

- 1 Apply with a cotton bud **0.5ml** of the solution at a **10cm x 10cm** area to be treated.  Amount of solution: **0.50ml**  Treatment area: **10cm x 10cm**
- 2 Choose the depth on **Revitalize Pen** to **0.75mm** and start work with it in straight lines horizontally and vertically at the area to be treated.  Depth: **0.75mm**  Speed: **fast** 
- 3 Apply another **0.5ml** of the solution at the same area.  Amount of solution: **0.50ml**  Treatment area: **10cm x 10cm**
- 4 Work with the **Revitalize Pen** in **circle** or **"infinity"** shapes covering all the area with the **same depth**.   Revitalize Pen depth: **0.75mm**

POST TREATMENT RECOMMENDATIONS

1. Do not touch the treated area.
2. Wear loose clothing over any treated area.
3. No makeup, skin preparations or abrasive products for 24 hours.
4. Avoid skin/hair preparations for 24 hours.
5. Expect swelling redness and bruising (these are temporary).
6. If you experience undue discomfort contact your practitioner immediately.
7. Expect some mild irritation and bumps under the skin (this is normal and will disperse).
8. Keep the treated area dry for 24 hours.
9. Avoid alcohol, exercise and extreme temperature for 48 hours.
10. Stay out of the sun (cover the area if going outside).
11. Avoid sun-beds, saunas and steam rooms.
12. Contact your practitioner if you have any concerns following treatment.

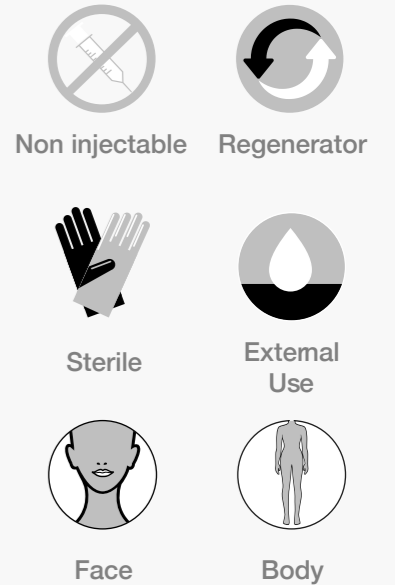

Following **Revitalize Booster** clinical trials microneedling techniques obtain the best aesthetic results with less trauma for the patient.

Revitalize Booster can be used with other mesotherapy techniques such as High Frequency (HiFu), Electroporation and Mesogun. Please note that other techniques may require a different setting. Refer to your training manual.

Precautions:

Topical use only, **DO NOT INJECT**. Avoid contact with eyes. In case of contact with eyes, rinse immediately with plenty of water. It is not recommended to use this product if you are allergic to any of its ingredients. Keep out of children's reach. Store under 25°C in a dry place

Do not expose to direct sunlight.

EXPLORE OUR COMPLETE MESOTHERAPY LINE:

A range of award winning serums:

Revitalize



www.myfiller.eu/revitalize